**THUASNE**

# MalleoDynastab® BOA®

<b>fr</b>	Chevillière d'immobilisation stabilisatrice avec système de serrage BOA Fit System .....	6
<b>en</b>	Stabilising ankle-immobilising brace with BOA Fit System closure .....	7
<b>de</b>	Stabilisierende Sprunggelenkorthese zur Ruhigstellung mit BOA Fit System zum Festziehen.....	8
<b>nl</b>	Stabiliserende immobilisatiebrace voor de enkel met BOA Fit System-klemsysteem .....	10
<b>it</b>	Cavigliera d'immobilizzazione stabilizzatrice con sistema di chiusura BOA Fit System .....	11
<b>es</b>	Tobillera de inmovilización estabilizadora con sistema de ajuste BOA Fit System .....	13
<b>pt</b>	Artelheira estabilizadora de imobilização com sistema de aperto BOA Fit System .....	14
<b>da</b>	Immobiliserende ankelstøttebind med BOA Fit System til stramning .....	15
<b>fi</b>	Immobilisoiva vakauttava nilkkatuki, jossa on BOA Fit System -kirstysjärjestelmä.....	17
<b>sv</b>	Stabiliserande och immobiliserande ankelortos med fästsystemet BOA Fit System .....	18
<b>el</b>	Σταθεροποιητική επιστραγαλίδα ακινητοποίησης με BOA Fit System .....	20
<b>cs</b>	Stabilizační ortéza na kotník se stahovacím systémem BOA Fit System .....	21
<b>pl</b>	Stabilizująca orteza skokowo-stopowa z systemem zaciskania BOA Fit System .....	22
<b>lv</b>	Stabilizējoša potītes imobilizācijas ortoze ar BOA Fit System .....	24
<b>lt</b>	Stabilizuojantis imobilizuojantis čiurnos įtvarytas su „BOA Fit System“ suveržimo sistema.....	25
<b>et</b>	BOA Fit System kinnitussüsteemiga stabiliseeriv fikseeriv hüppeliigese ortoos .....	26
<b>sl</b>	Opornica s stabilizatorjem za gleženj s sistemom vpenjanja BOA Fit System .....	28
<b>sk</b>	Imobilizačná členková ortéza s upínacím BOA Fit Systémem .....	29
<b>hu</b>	Stabilizáló bokarögzítő BOA Fit System szorítórendszerrel .....	31
<b>bg</b>	Обездвижващ стабилизатор за глезен със система за затягане BOA Fit System .....	32
<b>ro</b>	Glezniară stabilizatoare de imobilizare cu sistem de strângere BOA Fit System .....	33
<b>ru</b>	Ортез стабилизирующий голеностопный с системой фиксации BOA Fit System .....	35
<b>zh</b>	带BOA Fit System闭锁系统的稳定式护踝 .....	36
<b>ar</b>	رباط لتثبيت وحماية الكاحل ومزود بنظام ربط BOA Fit System .....	39

	0	16 - 19 cm
	1	19 - 22 cm
	2	22 - 26 cm
	3	26 - 30 cm

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Niikanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Hlezenní stabilizační ortéza

<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>zh</b>	脚踝尺寸
<b>ar</b>	الكلحاحل محيط








<b>fr</b>	Immobilisation	Stabilisation	Proprioception
<b>en</b>	Immobilization	Stabilization	Proprioception
<b>de</b>	Ruhigstellung	Stabilisierung	Propriozeption
<b>nl</b>	Immobilisatie	Stabilisatie	Proprioceptie
<b>it</b>	Immobilizzazione	Stabilizzazione	Propriocezione
<b>es</b>	Inmovilización	Estabilización	Propiocepción
<b>pt</b>	Imobilização	Estabilização	Proprioceção
<b>da</b>	Støtte	Stabilisering	Proprioception
<b>fi</b>	Immobilisoi	Stabiloi	Proprioseptio
<b>sv</b>	Immobilisering	Stabilisering	Proprioception
<b>el</b>	Ακινητοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
<b>cs</b>	Znehybnění	Stabilizace	Propriocepce
<b>pl</b>	Unieruchomienie	Stabilizacja	Propriocepcja
<b>lv</b>	Imobilizācija	Stabilizēšana	Propriocepcija
<b>lt</b>	Imobilizavimas	Stabilizavimas	Propriocepcija
<b>et</b>	Liikumatuks muutmine	Stabiliseerimine	Proprioceptsoon
<b>sl</b>	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
<b>sk</b>	Imobilizácia	Stabilizácia	Propriocepcia
<b>hu</b>	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
<b>bg</b>	Обезгвужване	Стабилизация	Проприоцепция
<b>ro</b>	Imobilizare	Stabilizare	Proprioceptie
<b>ru</b>	Иммобилизация	Стабилизация	Проприоцепция
<b>zh</b>	固定	稳定	本体感受
<b>ar</b>	الاستثبيت	الاستتقرار	الوعي الحسي استقبالي

<b>fr</b>	Appuyer pour enclencher	Tournez pour serrer	Tirez pour desserrer rapidement
<b>en</b>	Push in to engage	Turn to tighten	Pull up for quick release
<b>de</b>	Zum Festziehen eindrücken	Zum Festziehen drehen	Zum schnellen Lockern herausziehen
<b>nl</b>	Indrukken om aan te draaien	Draaien om aan te spannen	Optrekken om snel los te maken
<b>it</b>	Premere per inserire	Girare per stringere	Tirare per allentare
<b>es</b>	Empuje para armar	Gire para apretar	Levante para soltar rápidamente
<b>pt</b>	Premir para acionar	Rodar para apertar	Puxar para cima para soltar rapidamente
<b>da</b>	Skub ind for at fastgøre	Drej for at stramme	Træk op for en hurtig løsning
<b>fi</b>	Kytke painamalla	Kiristä kiertämällä	Vapauta nopeasti vetämällä
<b>sv</b>	Tryck in för att sätta fast	Vrid för att dra åt	Dra uppåt till för att lossa snabbt
<b>el</b>	Πιέστε προς τα μέσα για ασφάλιση	Στρέψτε για σύσφιξη	Τραβήξτε προς τα έξω για γρήγορη απελευθέρωση
<b>cs</b>	Stisknutím aktivujte	Otočením utáhněte	Zvednutím povolte
<b>pl</b>	Wcisnąć, aby wprowadzić	Przekręcić, aby zamocować	Pociągnąć w górę, aby szybko zwolnić
<b>lv</b>	Nospiediet, lai ieslēgtu	Pagrieziet, lai savilktu	Pavelciet, lai ātri atbrīvotu
<b>lt</b>	Paspauskite, kad sujungtumėte	Pasukite, kad suveržtumėte	Patraukite, kad greitai atlaisvintumėte
<b>et</b>	Sulgemiseks vajutage	Pingutamiseks keerake	Kiireks lahti tegemiseks tõmmake
<b>sl</b>	Pritisni, da se vklopi	Zavrti, da se vpne	Povlecí gor, da se hitro odklopi
<b>sk</b>	Zatlačením zaistite	Otočením utiahnite	Potiahnutím rýchlo uvoľnite
<b>hu</b>	Rögztéshez nyomja meg	Szorításához fordítsa el	Gyors kioldáshoz húzza ki
<b>bg</b>	Натиснете, за да поставите	Завъртете, за да затегнете	Издърпайте нагоре за бързо освобождаване
<b>ro</b>	Apăsăți pentru a cupla	Rotiți pentru a strânge	Trageți în afară pentru a elibera rapid
<b>ru</b>	Нажмите, чтобы перевести систему фиксации в рабочее состояние	Поверните, чтобы затянуть шнурки	Потяните, чтобы расстегнуть
<b>zh</b>	向内推动实现啮合	转动收紧	向上拉起实现快速释放
<b>ar</b>	اضغطوا على الصمولة	أديروها لشد الشراك	اسحبوا لاراء سريع

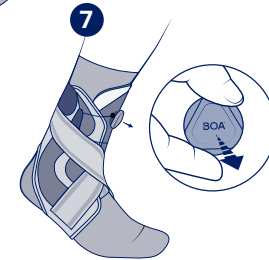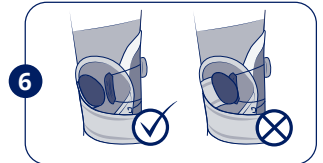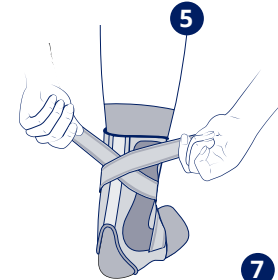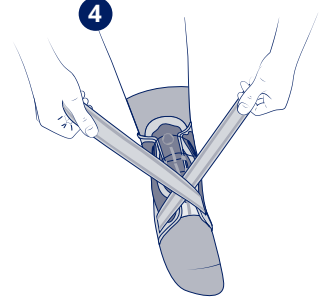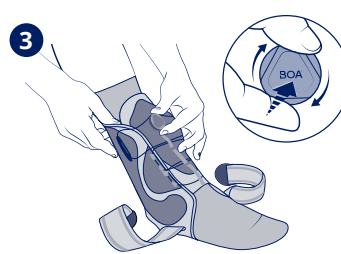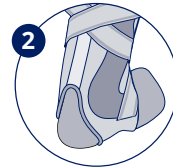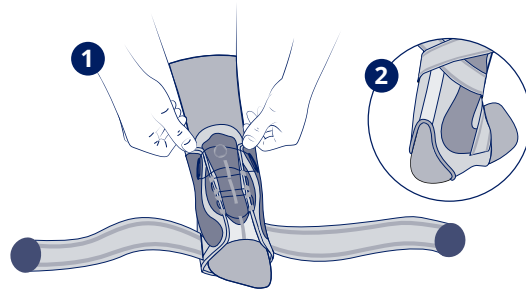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## en

# STABILISING ANKLE-IMMOBILISING BRACE WITH BOA FIT SYSTEM CLOSURE

## Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

## Composition

Textile parts: neoprene, polyamide, elastane, polyester, polyethylene, polyurethane.  
Elastic straps: polyester, elastane.  
Self-fastening tabs: polyamide.  
Plastic parts: polyamide 66.  
Side reinforcements: stainless steel.

## Properties/Mode of action

Stabilising ankle brace to provide immobilisation, incorporating rigid frame and elasticated straps to give stability and immobilisation to the ankle.  
Fast, durable and micro-adjustable BOA Fit System for a custom fit, with a lifetime guarantee  
A slim product that will fit easily inside a shoe.  
This is a bilateral product.

## Indications

Ankle immobilisation.  
Moderate to severe ankle sprain.  
Joint laxity/chronic instability prevention during sport activities resumption.  
Sport activities resumption.  
Tendinopathy treatment.  
Conservative treatment of Achilles tendon injuries.

## Contraindications

Do not use this product if the diagnosis has not been confirmed.  
Do not apply the product in direct contact with broken skin.  
Do not use in the event of known allergy to any of the components.  
History of venous or lymphatic disorders.

## Precautions

Verify the product's integrity before every use.  
Do not use the device if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart.  
It is recommended that a healthcare professional supervises the first application.  
Strictly comply with your healthcare professional's prescription and recommendations for use.  
For hygiene and performance reasons, do not re-use the product for another patient.  
Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Do not wear the product in a medical imaging device.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral straps. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

#### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of vein thrombosis.

Any serious incident arising that is linked to the product should form the subject of a report to the manufacturers and the competent authority for the Member State in which the user and/or patient is domiciled.

#### **Instructions for use/Application**

We recommend that the product should be worn inside a shoe by daytime whilst walking, and by night for protection in order to prevent and limit sudden movements.

Before putting the ankle brace on, make sure the laces are completely unfastened.

##### A. Fitting the ankle brace:

Slide the foot inside the ankle brace. Ensure that the heel is resting inside the opening provided at the rear of the product for this purpose. ❶ ❷

##### B. Fastening the ankle brace:

Tighten the ankle brace using the BOA Fit System closure system; to do this, push the wheel in until you hear a 'click', then turn clockwise until the correct level of support is obtained. ❸

##### C. Fitting the straps:

Take a strap in each hand, then pass them over each other across the upper foot. ❹

(Fig. 4) Wrap around the ankle, cross the two straps over again around the rear of the heel, and attach the self-fastening tabs to the upper portion of the ankle-brace fabric. ❺

Verify the secure grip of the self-fastening tabs over the fabric area by pressing on the self-fastening part. Do not place the self-fastening tabs over the black plastic portion. ❻

##### D. Releasing the ankle brace:

Pull on the wheel until you hear the 'click' to unfasten the laces. ❼

##### E. Fitting inside a shoe:

Once the ankle brace is in position, slide the foot carefully inside a closed shoe and fasten the shoe's closure mechanism. ❽

In the event of any discomfort at the ankle-bone area, it is possible to change the position of the stays: Detach the self-fastening tabs from the black fabric area, take hold of the rigid metal part and remove it. Slide the stays above or below the stitching located either side of the ankle brace. ❾

#### **Care/Maintenance**

Product can be washed in accordance with the conditions shown on this notice and on the label. Make sure to remove the lateral stays before washing. Replace in the original location before next use. Close the self-fastening tabs and close the product (BOA Fit System) before washing. If possible, use a washing net.

Dry in a ventilated area, away from sources of heat.

#### **Storage**

Store at room temperature, preferably in its original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

#### **Warranty**

BOA Technology Inc. guarantees the closure system (wheel and lace) for the product's lifetime. This warranty applies for normal conditions of use of the product.

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©Thuasne - 2019302 (2022-05)